



Double Bed Raisers— Instruction sheet

Installing the raiser

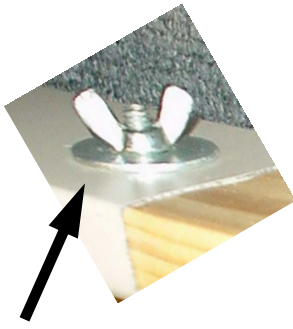

(1) The raiser will expand out in length. First undo the wing nut that clamps the raiser in a fixed position. The wing nuts undo in an anti clockwise direction .

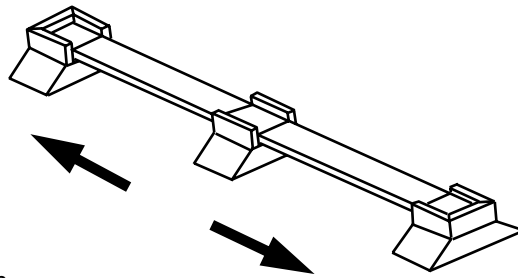

(2) Expand the raiser out to a size just a little bigger than the sofa or bed base.

(3) Slide the raiser under. Adjust the raiser to fit against the legs and tighten the wing nut in a clockwise direction.



(4) Check that all legs are positioned safely in the corners of the raiser. Sit on the sofa or bed and check that it feels safe to you .

Trial equipment

If you have this equipment on trial and it does not suit your client (for what ever reason) we are only too happy to have it back.

- If unsuccessful the trial equipment must be returned in original condition within 21 days, in order to get a full credit.
- Return the equipment to : **Multifit, 10 Pohutu Street, Whakatane.**
- Please be aware that you will have to pay the return freight charges.
- Please advise us if you trial with another client.
- If the trial is successful payment will be due on the 20th of the month following

Cleaning

All surfaces are pre finished and only require dusting occasionally. Reissue clean up can be done with any mild detergent, marks can be removed with Jiff.

Invoicing

Invoices will arrive in the post within a day or two of the product delivery.

Parts replacement

All parts are available by faxing us at 0800 102 800

Guarantee

- Delivery : Delivery is within 5 working days, or we will shout your whole OT dept morning tea.
- Will it fit : This equipment will fit or we will customize one that will.
- How long will it last : This equipment will retain its original function for at least 2 years or we will replace it free of charge.

Reusability / cost effectiveness

We have tracked the reuse of equipment over the years. Original Multifit chair raisers made in 1990 are still in good service today.

Optional extras

Chair side transfer rail : Available in left or right side of the chair and as a double rail.

Adjustable height : Available in height adjustable option as well as made to measure .

Research and development

Multifit equipment is unique to our company, because all our designs are developed through feed back from NZ Occupational therapists and only sold through our business. We welcome your feed back, so that we are able to resolve any frustrations that you have.

Web site (www.multifit.co.nz)

Keep a look out for new products and information on our ever expanding range of unique products, as well as our news letter for updates and inspiration.

In service Demonstrations and training

Available on request. Wayne likes to get around and see the OT's every 6 months (includes the Show your ability expo in March). If you would like an in service, please phone 0800 102 803 and we will book you in.

Safety first

This equipment is made with the best materials, workmanship, and design. As chair designs are many and varied, it is up to the person fitting the chair raiser, to determine its suitability for the chair you intend to raise. We have chair raisers made to suit most chairs, so make sure you order the right raiser for the right chair. If you have any doubt , please contact us and we will either send you one of our purpose made raisers or make a raiser up especially for you.

Contact details

Multifit Hospital Supplies. Ph 0800 102 803—Fax 0800 102 800 - E Mail info@mutifit.co.nz -
P O Box 748 - 10 Pohutu, Street, Whakatane

Quality of life is precious. Our mission is :

To provide you with useful, quality, and long life equipment, that enables your clients to make the most of their lives.

Serving the people with disabilities since 1990