

Bed lever STD MF 3/1 STD

50% stronger & safer design, means your client is protected from injury as well as getting a more steady rail.

50 % stronger because we have increased the base board thickness and length so you get minimal flexing of the base board and therefore a more steady rail . Added to that we have actually welded the rail to the base board for added strength.

Safer design because the joint between the rail and the rail base is of slim design and all the edges are rounded your client is protected from injury that they might have gotten from a more bulky design.

Increases your clients independence.

A steady hand hold beside the bed can be used to assist your client in and out of their bed . By having something to hold onto, as they are sitting up, trying to roll over or when getting out of their bed can make all the difference when they are trying to do it unaided .

How you can fit the bed rail The base board of the rail fits between the mattress and the bed base. The rail extends up the side of the bed . Its purpose is to provide a hand hold

assisting your client in and out of bed.

Will not pull out from the bed

Because the bed rail can be fixed to the bed with a strong braided cord, when your client pulls on the rail, you can be sure that rail will not pull out from their bed. This feature gives the bed rail a very solid feel for your client and the security of knowing that they are safe.

New design

Now with New improved base.



Std Bed lever with cord & cleat fixing

Optional

double handled rail improves performance



Double handle option , has a second horizontal bar giving your client a lower handhold that is useful to grasp when they wish to turn over.

- Use our **SAFE RAIL** if entrapment is an issue.

Independently tested to confirm strength and stability

Specifications :

- The base board is made from 16 mm fibre board to give you strength.
- Long lasting because we only use quality materials, so you know that our rails can be used over and over again and still look good.
- All the fibre board is furniture grade so you get good quality
- Clear finished gives you long lasting good looks .
- Braided cord & cleat fixing spans the width of the bed, clamping to the far side ensuring that the rail stays in place when your client pulls on the handle, so you know it's safe.
- Beefy construction ensures strength and safety for your client .
- Bed rail : Tube is powder coated, size is 25 mm, hoop size 550 mm high x 235 mm wide

14 Day trial Without obligation to purchase.

Delivery is fast Delivery is normally within 5 working days

Ordering is easy We have a free fax number 0800 102 800, so it costs you nothing to contact us. Just jot down your name, client & delivery address. We would love to hear from you .

Protect your client from injury



The low profile joint between the rail and the rail base means no bulky sharp edges extend from the bed, and therefore your client is protected from injury

Guarantee

- Delivery with **in 5 working days** or we will shout your whole OT department morning tea .
- The equipment will retain its original function for **at least 2 years** or we will replace it **free of charge**

- Customised & developmental equipment may take a little longer.

Bed rail MF 3/1 std – Instruction sheet

Installing the bed rail



- (1) Slide the base of the bed rail between the mattress and the base of the bed.
- (2) Position the handle to the most convenient location for your client. Usually about 600 mm down from the head end of the bed.



- (3) **Braided cord** : Extend the braided cord to the other side of the bed, pull the cord tight through the block and secure the cord in the plastic cleat.
- (4) **Strap and buckle** : Extend the strap between the mattress and the bed base to the other side of the bed. Secure the end of the strap around the bed leg and tighten the strap

Trial equipment

If you have this equipment on trial and it does not suit your client (for what ever reason) we are only too happy to have it back.

- If unsuccessful the trial equipment must be returned in original condition within 14 days, in order to get a full credit.
- Please be aware that you will have to pay the return freight charges.
- Please advise us if you trial with another client.
- If the trial is successful payment will be due on the 20th of the month following

Cleaning

All surfaces are pre finished and only require dusting occasionally. Reissue clean up can be done with any mild detergent, marks can be removed with Jiff.

Invoicing

Invoices will arrive in the post within a day or two of the product delivery.

Parts replacement

All parts are available by faxing us at 0800 102 800. (although replacing parts on this product has never been required.)

Guarantee

- Delivery : Delivery is within 5 working days, or we will shout your whole OT dept morning tea.
- Will it fit : This equipment will fit or we will customize one that will.
- How long will it last : This equipment will retain its original function for at least 2 years or we will replace it free of charge.

Reusability / cost effectiveness

We have tracked the reuse of equipment over the years. Original Multifit Bed levers made in 1990 are still in good service today.

Research and development

Multifit equipment is unique to our company, because all our designs are developed through feed back from NZ Occupational therapists and only sold through our business. We welcome your feed back, so that we are able to resolve any frustrations that you have.

Web site (www.multifit.co.nz)

Visit our website for information on our ever expanding range of unique products, as well as our news letters and inspiration.

In service Demonstrations and training

Available on request. Wayne likes to get around and see the OT's every 6 months (includes the Show your ability expo in March). If you would like an in service, please phone 0800 102 803 and we will book you in.

Safety first

This equipment is made with the best materials, workmanship, and design. If your client has difficulty using the bed rail, or would prefer a rail that moves less, we have other styles that you might like to try.

Multifit Hospital Supplies Ltd - Phone 0800 102 803 - Fax 0800 102 800, - E Mail info@multifit.co.nz

Returning equipment : 10 Pohutu Street, Whakatane

*Quality of life is precious. Our mission is :
To provide you with useful, quality, and long life equipment, that enables your clients to make the most of their lives.
Serving the people with disabilities since 1990*