

Big Softie Chair Raiser MF 1/7



Fig 1

(1) Perimeter lip (2) Wing nuts
(3) Knobs (4) Cord & Cleat (5) Splayed feet

For Big Softie Chairs that rock or recline, this Raiser is a perfect solution.

Splayed feet give extra stability to chairs with a narrow base like this one.

Two strong cord and cleats resist the chair jumping off the Raiser as the user rocks back and forth.

- **Chair stays in place** - 20mm perimeter lip holds the Raiser to the chair.
- **Rocking the chair is safe** - cord and cleat hold the chair to the Raiser.
- **Extra stability** is provided with splayed feet
- **Easy to fit** - Telescopic adjustment ensures you get the perfect fit easily
- **Tested** - Independently tested to ensure your safety.
- **Unique clamping action** - holds the Raiser tightly to a fixed size.
- **Re Issuable** - Robust construction enables the Raiser to be re issued.
- **Cost effective** - Raisers sold 25 years ago are still in service today, making them very long lasting and cost-effective.

SPECIFICATIONS

Max user weight		380Kg (+ the furniture)		
lip around the perimeter		20mm		
BigSoftie Raiser Chair Platform				
Height	Size	Supplier Code	Order Code	JDE#
150mm	490 x 528 mm to 705 x 880 mm	MF 1/7-150	K13-150	760413
100mm		MF 1/7-100	K13-100	760405
75mm		MF 1/7-75	K13-75	760430
50mm		MF 1/7-50	K13-50	760421
Other options - Custom made.				
Optional: non slip rubber pads stop raiser slipping on polished timber floors				
Safety First - Raiser must match the chair type. See Www.multifit.co.nz				
Tested - to ensure the stability of the raised furniture. This safeguards the user.				

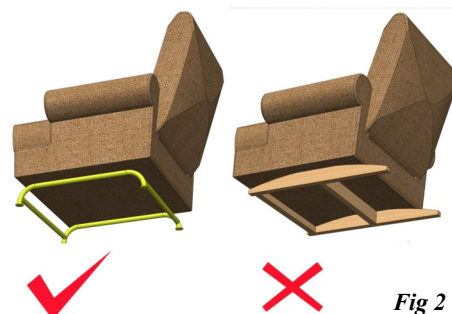


Fig 2

Raiser will fit this type of chair

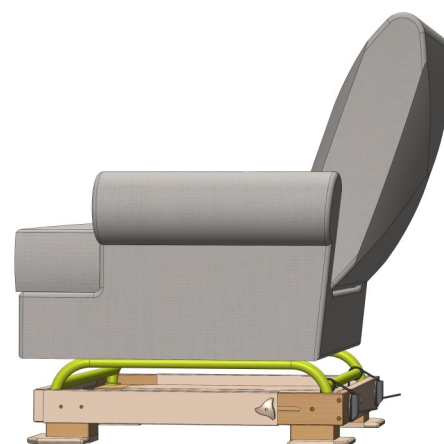


Fig 3

Raiser fitted to a Big Softie chair

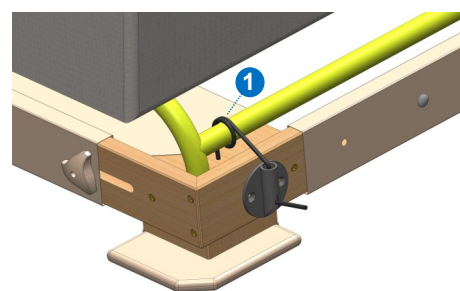


Fig 4

(1) The rear of the chair is held to the Raiser by use of a cord that wraps around the base of the chair and down through a nylon cleat.

Guarantee

- You will receive any List Item within five working days or we will treat you morning tea.
- Our equipment will fit or we will customise one for you that will.
- Our equipment will retain its original function for at least two years or we will send you a replacement at no extra cost.



Made with pride
In New Zealand
for over 25 years



Multifit Hospital Supplies Ltd © 2020

Ph: 0800 102 803, E: reception@multifit.co.nz

Website: www.multifit.co.nz

170720

Big Softie Chair Raiser MF 1/7


Fitting Instructions




MultiFit

Get in & out of your chair with less effort.


The following method ensures the Raiser is fitted correctly, safely and with the minimum of effort.

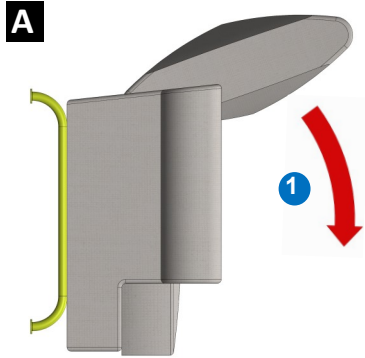


(1) Roll the chair forward so the feet of the chair are accessible.

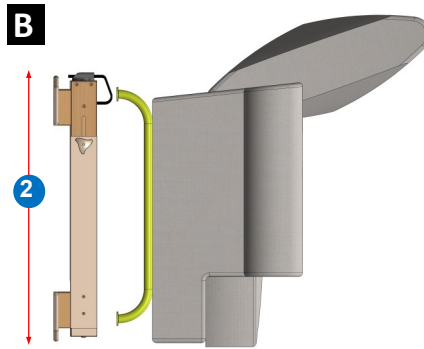


(2) Adjust the Raiser in width and length so the lip of the Raiser fits snugly over the base of the chair.

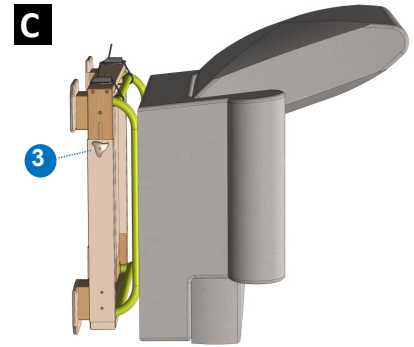

(3) Tighten the knobs & wing nuts on all four telescopic sides of the Raiser.

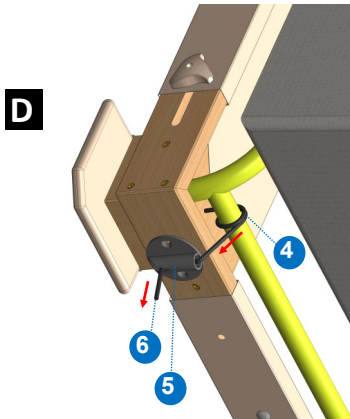

(4) loop the cord over the tube of the chair base, then around & down through the cleat (5)
(6) Pull the cord out at 90 degrees to the cleat. This will lock the cord in the wedge of the cleat so no knot is required.

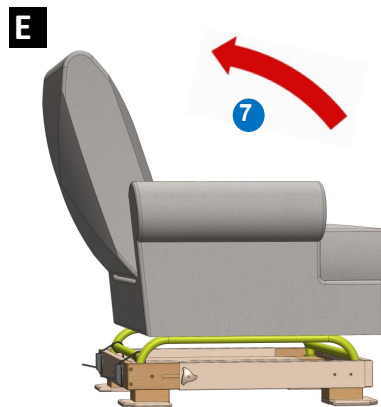

(7) Roll the Chair Raiser back onto its base and push the raised chair back into its original location.

Checks:

Confirm the raised chair is safe and stable.
When getting out of the chair your feet are flat on the floor and your bottom approximately level with the underside of your knees.
Is getting in and out of the chair easier?
Is the comfort acceptable?



Assessment notes: Was the chair raised to the correct height? Does the user look more confident? Chairs are most stable when getting in and out. Reaching over the side of the chair is not recommended. If over time the situation changes and the user needs additional assistance, consider Chair Transfer Rails or an orthopaedic chair.

Maintenance - check monthly the tightness of the raiser to the chair, tightness of wing nuts, knobs and fixings.

Cleaning - with damp cloth and non-abrasive detergent, clean and dry thoroughly.

Demonstrations - see our stand at the next Show Your Ability expo.

Parts replacement - refer to www.multifit.co.nz for a complete parts list.

Our Mission - is to provide you with the safest, most cost-effective raiser to meet your needs.



Scan for more info



Made with pride
In New Zealand
for over 25 years



MultiFit Hospital Supplies Ltd © 2020
Ph: 0800 102 803, E: reception@multifit.co.nz
Website: www.multifit.co.nz 170720