




MF BTR Insert & installation instructions

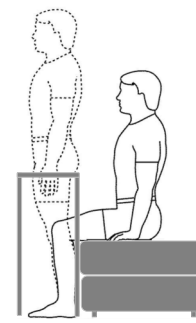

Get in & out of your bed with less effort.

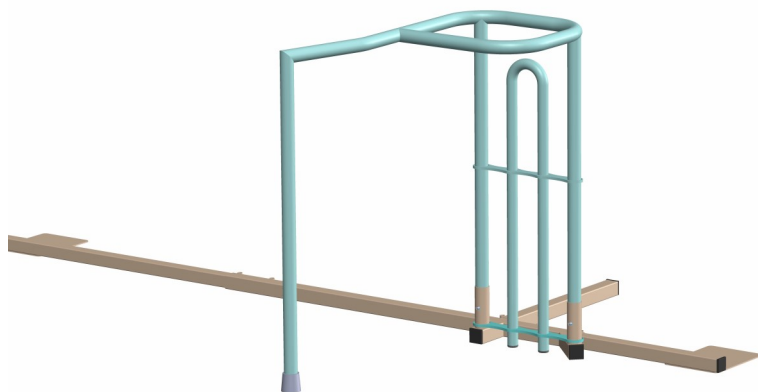


The BedSide Transfer Rail with Insert fitted

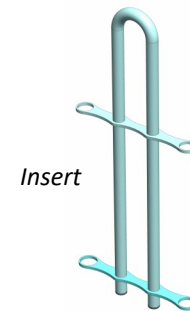


Insert

Safety First To resist head traps, older models of the BedSide Transfer Rail can easily be retrofitted with an Insert by following steps A,B,C


- Ensure the Rail is positioned against the mattress so no entrapment can occur between the mattress and the rail
- Not for use with electrically height adjustable beds
- Not for use with children

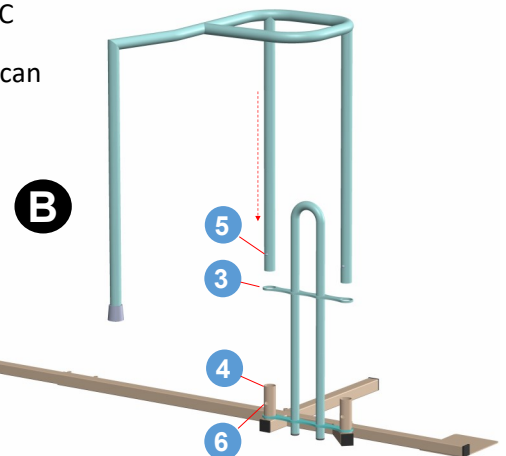

Replace the Handrail by sliding it through the Top Loops of the Insert (3) and down into the Base Tubes (4) so the Spring Clips (5) pop into the holes of the Base Tube (6)

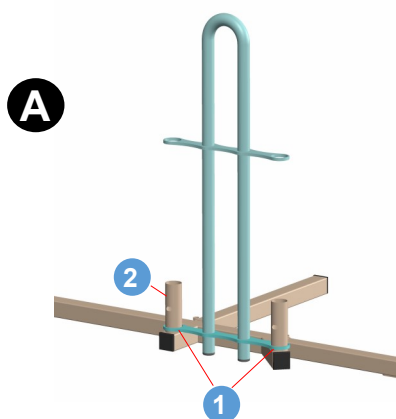

With the handrail removed, slide the lower loops of the Insert (1) over the Base Tubes (2)

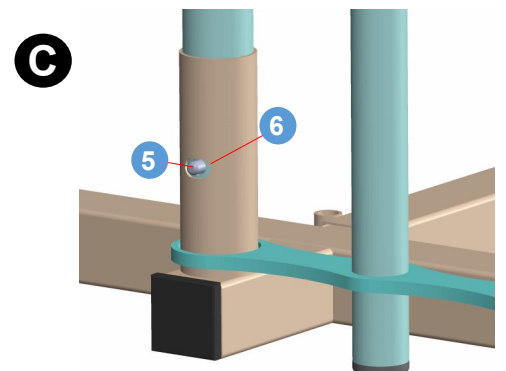

SPECIFICATIONS

Max user weight	130Kg
Distance Rail extends from the bed	600mm
Rail Height	810mm
Rail Diameter	32mm
Can be used with any bed which has a gap of 35mm between the bed and floor	
NOT FOR USE WITH HI /LOW PROFILE HOSPITAL BEDS	



Made with pride
In New Zealand
for over 25 years



Multifit Hospital Supplies Ltd © 2020,
Ph: 0800 102 803, E: reception@multifit.co.nz
Website: www.multifit.co.nz 090720