

Double Bed Raiser MF 2/4

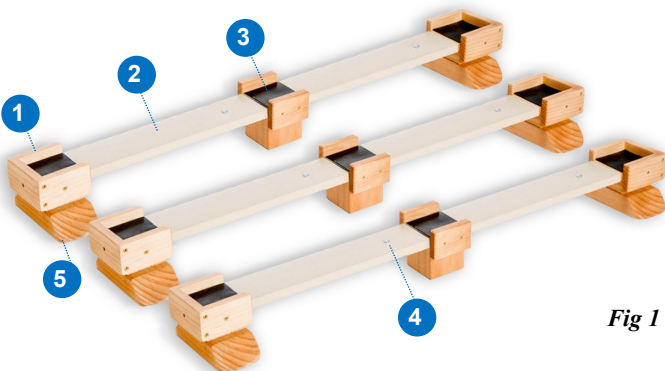


Fig 1

(1) Perimeter lip (2) Telescopic Section (3) 90x90 Pads
(4) Bolt & wing nut (5) splayed feet

For raising a double bed, this is a great solution.

They are easier and quicker to fit than individual bed blocks. You will also get increased stability & safety of the raised bed.

The Double Bed Raiser pads are connected with a telescopic section that expands easily and clamps rigidly to the perfect length for your needs.

- Easy to fit
- Extra stability
- Unique clamping action
- Safe
- Re Issuable
- Cost effective


SPECIFICATIONS

Max user	380Kg	Adjusts by hand—no tools required
Perimeter lip	20mm	Tested to resist overbalancing of the bed
Pad size	90 x 90mm	Optional: Non slip rubber under the feet.

Fits Beds: Double, Queen, King, Super King (California King custom made)



Double Bed Raiser Set of 2

Height	Size	Supplier Code
150mm	1150	MF 2/4-150 / 2
100mm	mm to	MF 2/4-100 / 2
75mm	1700	MF 2/4-75 / 2
50mm	mm	MF 2/4-50 / 2

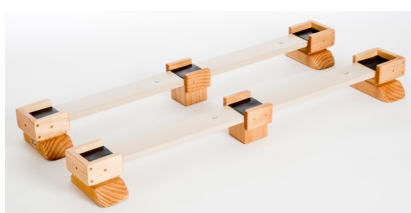

Double Bed Raiser Set of 3

Height	Size	Supplier Code
150mm	1150	MF 2/4-150 / 3
100mm	mm to	MF 2/4-100 / 3
75mm	1700	MF 2/4-75 / 3
50mm	mm	MF 2/4-50 / 3

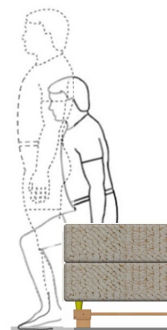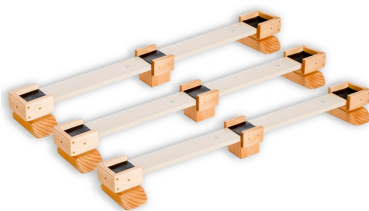

Pg1



MultiFit

Get in & out of your bed with less effort.



Fig 2

Raisers will fit this type of bed




Raisers fitted to double bed

Fig 3



Fig 4


Double Bed Raisers fitted to the headboard (1) of a double bed

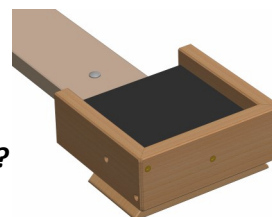

LARGER FEET?

Choose the
MF 2/4W Widened
Double Bed Raiser

Fig 5

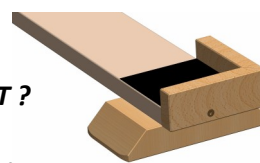

SMALLER FEET ?

Choose the
MF 2/4D Divan
Double Bed Raiser

Fig 6

Guarantee Our equipment will fit or we will customise that will.



Made with pride
In New Zealand
for over 25 years



Multifit Hospital Supplies Ltd © 2020

Ph: 0800 102 803, E: reception@multifit.co.nz

Website: www.multifit.co.nz


240720

Double Bed Raiser MF 2/4

Fitting Instructions




The following method ensures the Raiser is fitted correctly, safely and with the minimum of effort.

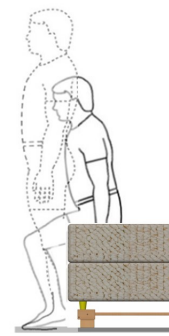

Pg2

MultiFit


Get in & out of your bed with less effort.

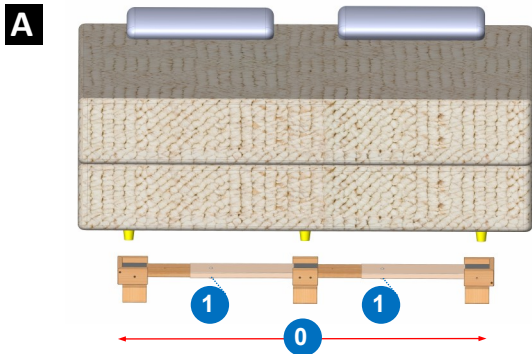

At the end of the bed, adjust the Raiser (0) in length, so the feet of the bed fit within the lip of the Raiser. Then tighten the wing nut bolts (1)




Using the Multifit Bed Jack (6) lift the end of the bed up. Fit the last Raiser (7) under the feet of the bed.

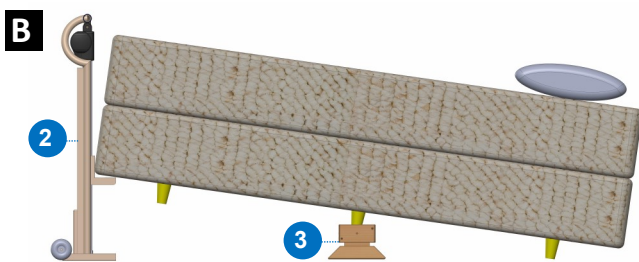

Using the Multifit Bed Jack (2) lift the end of the bed up. Fit one of the Raisers (3) centrally under the middle feet of the bed.



Place a weight (4) to tilt the head end of the bed upward. Fit the second Raiser (5).

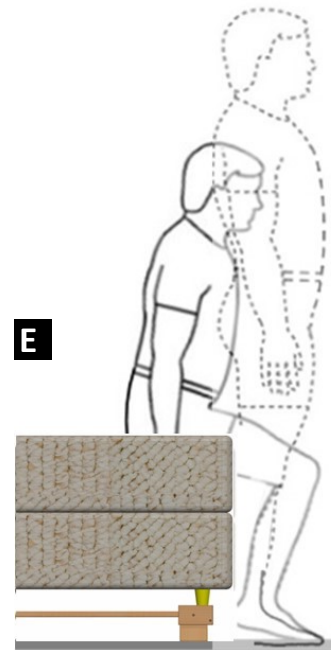

Check: Confirm the raised bed is safe and stable.
Is getting in and out of the Bed easier?
Was the bed raised to the correct height?
Does the user look confident?

Maintenance - check monthly the raised bed stability. Repeat steps A,B,C,D,E if required.

Cleaning - with damp cloth and non-abrasive detergent, clean and dry thoroughly.

Demonstrations - see our stand at the next Show Your Ability Expo.

Parts replacement - refer to www.multifit.co.nz for a complete parts list.

Our Mission - is to provide you with the safest, most cost-effective raiser to meet your needs.



Scan for more info



Made with pride
In New Zealand
for over 25 years



MultiFit Hospital Supplies Ltd © 2020
Ph: 0800 102 803, E: reception@multifit.co.nz
Website: www.multifit.co.nz, 240720