

# Open Front Raiser MF 1/OF

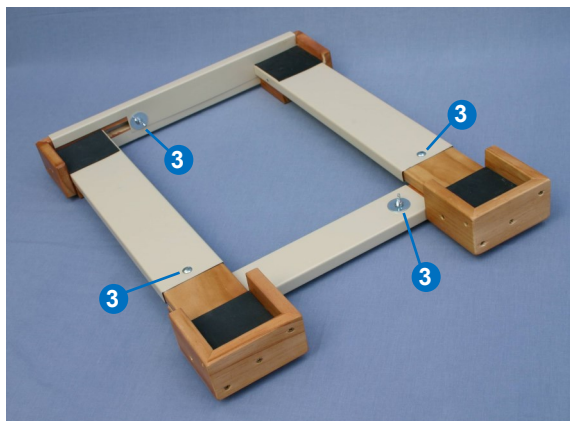


Fig 1

The front edge of this raiser is recessed, which allows you to get your feet further under the edge of your chair. This means that you don't have to slide as far forward to get your body weight over your feet to stand. You can also make better use of the arm rests on the chair.

## Fitting Instructions.

Extend the Raiser in width and length to suit the base of your chair (1- fig 2)

Adjust the Raiser so that it fits tightly up to the frame of your chair (2 - fig 3)

Tighten the nuts that secures the adjustment of the raiser (3 - fig 1)

Finally Test for safety and stability before use.

## Specifications

Max user weight	380Kg (+ the furniture)
Pad Size	90 mm square
lip around perimeter	20mm
Adjustment	<b>Length</b> 620-905mm <b>Width</b> 500-700mm

5 Leg Raiser Chair Platform			
Height	Supplier Code	Order Code	JDE#
150mm	MF 1/OF-150		
100mm	MF 1/OF-100		
75mm	MF 1/OF-75		
50mm	MF 1/OF-50		

**Optional:** non slip rubber pads stop raiser slipping on polished timber floors

**Safety First** - Raiser must match the chair type. See [www.multifit.co.nz](http://www.multifit.co.nz)

**Tested** - to ensure the stability of the raised furniture. This safeguards the user.



Fig 2

Raiser will fit this type of chair



Fig 3

Adjust the raiser to fit your chair.

**Assessment notes:** Was the chair raised to the correct height? Does the user look more confident? Chairs are most stable when getting in and out. Reaching over the side of the chair is not recommended. If over time the situation changes and the user needs additional assistance, consider Chair Transfer Rails or an orthopaedic chair.

**Maintenance** - check tightness of the raiser to the chair, tightness of wing nuts, knobs and fixings.

**Clean** - with damp cloth. Use non-abrasive detergent and dry thoroughly.

**Demonstrations** - see our stand at the next Show Your Ability Expo.

**Parts replacement** - refer to [www.multifit.co.nz](http://www.multifit.co.nz) for a complete parts list.

**Our Mission** - is to provide you with the safest, most cost-effective raiser to meet your needs.



Made with pride  
In New Zealand  
for over 25 years



**MultiFit Hospital Supplies Ltd** © 2020  
Ph: 0800 102 803, E: [reception@multifit.co.nz](mailto:reception@multifit.co.nz)  
Website: [www.multifit.co.nz](http://www.multifit.co.nz) 180720