

# Pedestal Chair Raiser MF PED

Pg1



**MultiFit**

Get in & out of your chair with less effort.

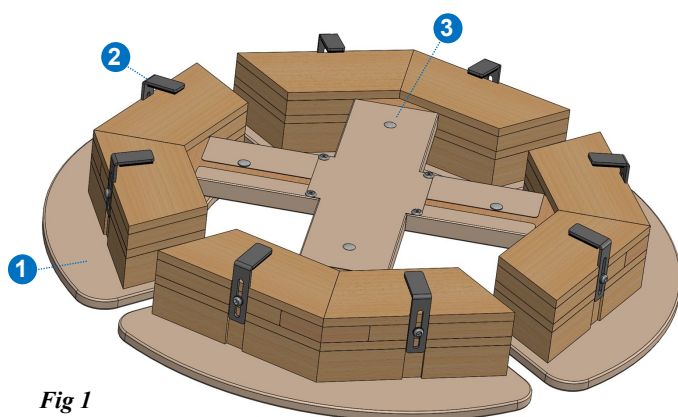


Fig 1

**The MF PED fits to chairs which have a round base.** The diameter expands, allowing metal clips to hook over the edge of the chair base, holding the raiser safely to the chair.

- (1) Splayed foot - adds stability.
- (2) Perimeter clip - attaches to the chair base.
- (3) Bolt clamp for Telescopic center - adjusts and fixes raiser diameter.

Pedestal Chairs come in two basic types.

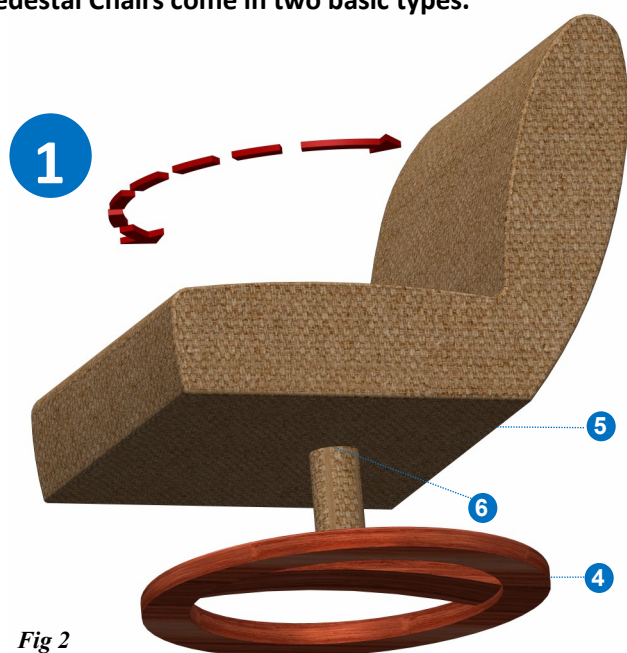


Fig 2

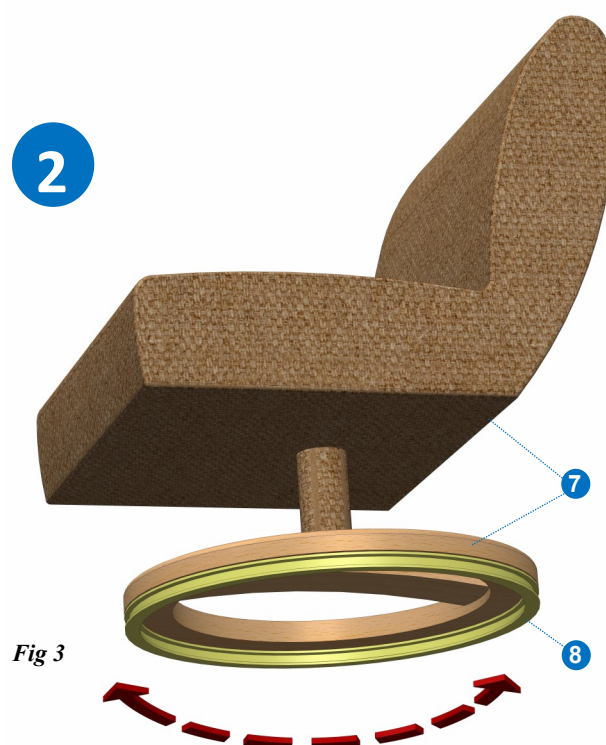


Fig 3

**Chair Type 1 - The base is fixed and only the seat turns.**

The base (4) does not turn. The seat (5) turns on a swivel (6). This type of chair requires the long hook fixing clips 9. (Fig 4 & Fig 7).

**Chair Type 2 - The base and seat turn on a swivel ring.**

Seat and base (7) turn on a swivel ring (8) located underneath the base. This requires the short hook fixing clips 10 or 11. (Fig 4 & Fig 8).

## SPECIFICATIONS

<b>Max user weight</b>	380Kg (+ the furniture)
<b>Diameter adjustment</b>	535 - 735 mm
<b>Optional:</b> rubber pads prevent slipping on timber floors	
<b>Safety First</b> - Raiser must match the chair type.	
Height	Supplier Code
150mm	MF PED -150
100mm	MF PED -100
75mm	MF PED-75
50mm	MF PED-50



Fig 4

Clips are made in a variety of shapes to suit different applications.




Made with pride  
In New Zealand  
for over 25 years



**MultiFit Hospital Supplies Ltd** © 2020,  
Ph: 0800 102 803, E: reception@multifit.co.nz  
Website: www.multifit.co.nz 050321

# Pedestal Chair Raiser MF PED

## Fitting Instructions

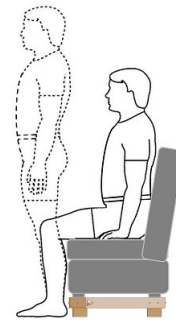

Pg2

**MultiFit**

Get in & out of your chair with less effort.

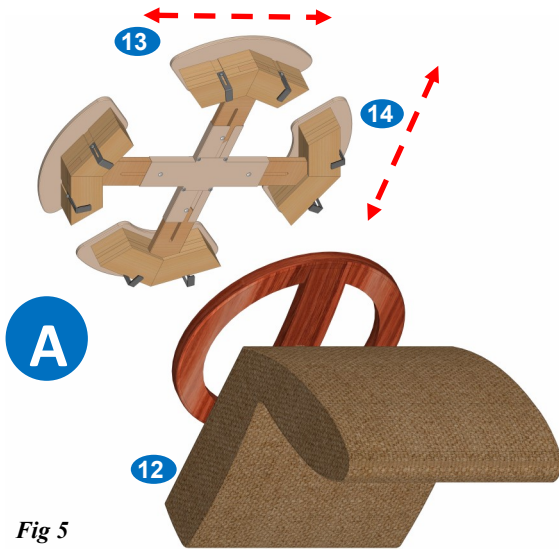


Fig 5

Turn the chair over (12). Expand the PED raiser out (13&14).

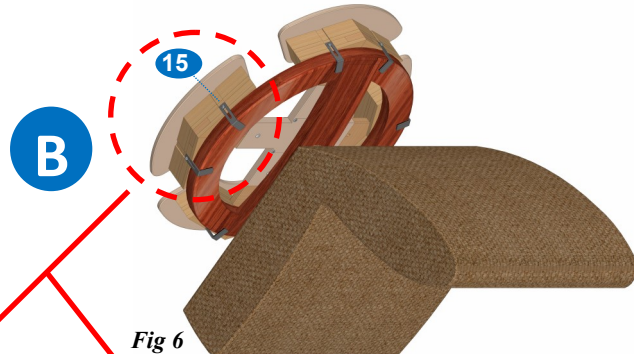


Fig 6

Fit the PED raiser over the base of the chair so that the clips (15) fit appropriately over the base of the chair.

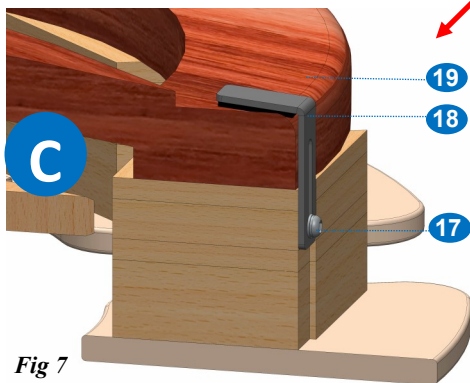


Fig 7

**Chair Type 1** Using the Allen key supplied (16) loosen adjustment bolt (17) adjust Clip (18) so it sits snugly on the top surface of the chair base (19) Tighten bolt (17).

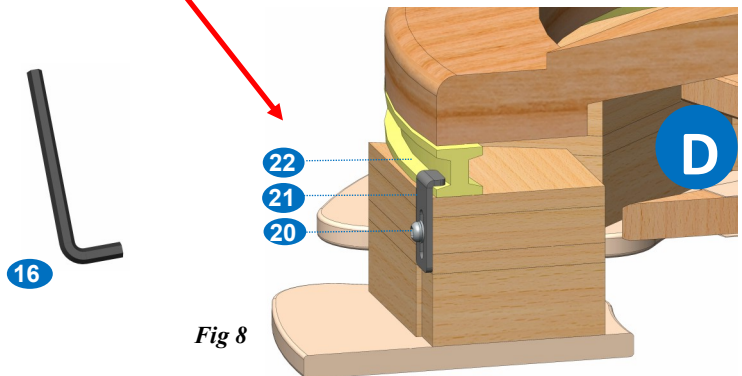


Fig 8

**Chair Type 2** Using the Allen key (16) loosen adjustment bolt (20) adjust Clip (21) so that it sits snugly on the lower edge of the swivel ring (22) as shown in fig 8 above. Tighten bolt (20).

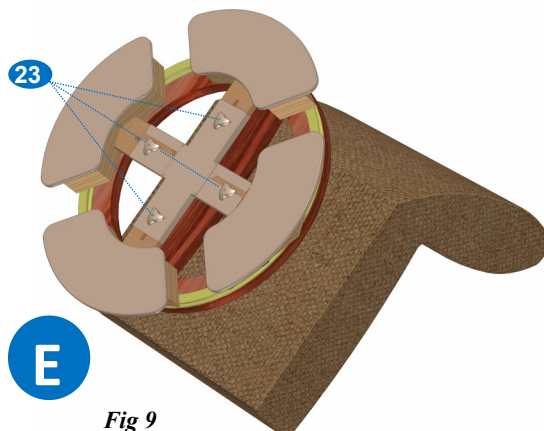


Fig 9

Tighten all 4 wing nuts (23) Check the Clips in the previous procedure remain correctly fitted.

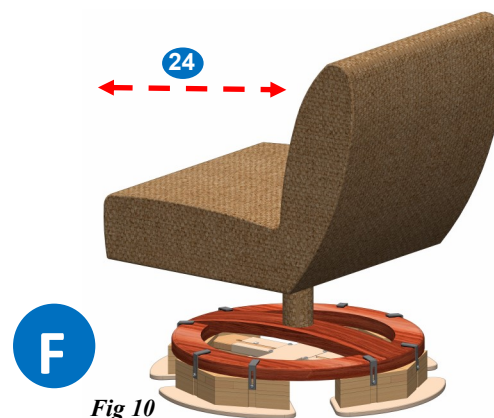


Fig 10

Turn the chair right way up. Sit on the chair and test (24) that all is safe.



Scan for more info



Made with pride  
In New Zealand  
for over 25 years



**MultiFit Hospital Supplies Ltd** © 2020  
Ph: 0800 102 803, E: reception@multifit.co.nz  
Website: www.multifit.co.nz 050321