

# Single Bed Raiser MF 2/2

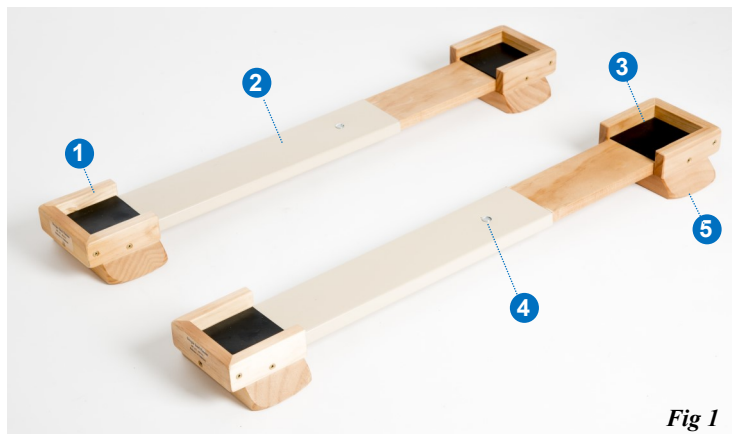


Fig 1

(1) Perimeter lip (2) Telescopic Section (3) 90x90 Pads  
(4) Bolt & Wing nut (5) Splayed feet

**For raising a single bed, this is a great solution.** It is easier and quicker to fit than individual bed blocks. You will also get increased stability & safety of the raised bed.

The blocks of the Single Bed Raiser are connected with a telescopic section that expands easily and clamps rigidly to the perfect length for your needs.

- Easy to fit
- Extra stability
- Unique clamping action
- Safe
- Re Issuable
- Cost effective

## SPECIFICATIONS


<b>Max user</b>	380Kg	<b>Optional:</b> Non slip rubber feet
<b>Perimeter lip</b>	20mm	<b>Tested</b> - to ensure stability
<b>Pad size</b>	90 x 90mm	Raiser must match Bed type

### Single Bed Raiser Set of 2

Height	Size	Supplier Code	
150mm	624 mm	MF 2/1-150 / 2	
100mm	mm to 900	MF 2/1-100 / 2	
75mm		MF 2/1-75 / 2	
50mm		MF 2/1-50 / 2	

### Single Bed Raiser Set of 3

Height	Size	Supplier Code	
150mm	624 mm	MF 2/1-150 / 3	
100mm	mm to 900	MF 2/1-100 / 3	
75mm		MF 2/1-75 / 3	
50mm		MF 2/1-50 / 3	

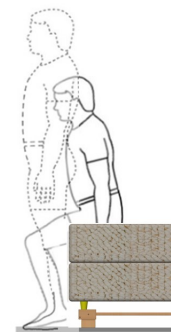

Pg1



## MultiFit

Get in & out of your bed with less effort.



Fig 2

Raiser will fit this type of bed

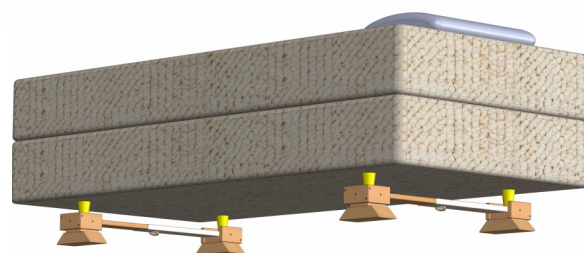


Fig 3

Raiser fitted to single bed

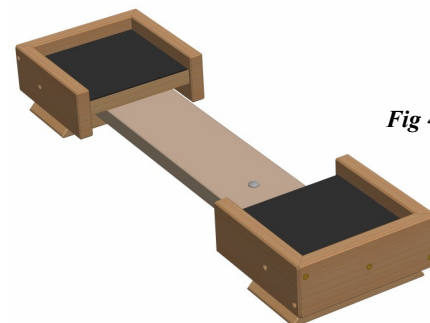


Fig 4

**LARGER FEET?** - if the feet of your bed are larger than 90mm then choose the **MF 2/2W** Widened Single Bed Raiser

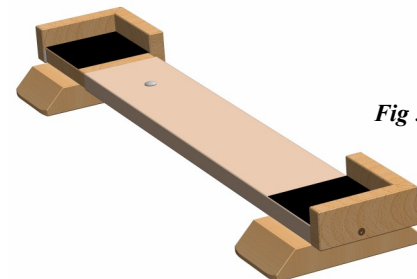


Fig 5

**SMALLER FEET ?** - if the feet are shorter than 20mm or there are no feet at all then choose the **MF 2/2D** Divan Single Bed Raiser

**Guarantee** Our equipment will fit or we will customise that will.




Made with pride  
In New Zealand  
for over 25 years



**MultiFit Hospital Supplies Ltd** © 2020  
Ph: 0800 102 803, E: reception@multifit.co.nz  
Website: www.multifit.co.nz 230720

# Single Bed Raiser MF 2/2

## Fitting Instructions

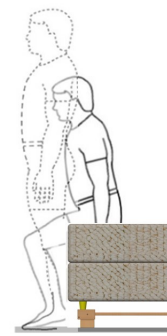

Pg2

**MultiFit**

*Get in & out of your bed with less effort.*

**A**



At the end of the bed, adjust the Raiser (0) in length, so the feet of the bed fit within the lip of the Raiser. Tighten the wing nut (1)

**B**




Using the Multifit Bed Jack (2) lift the end of the bed up. Fit one of the Raisers (3) centrally under the feet of the Bed.

**C**



Place a weight (4) to prevent the bed tipping. With the Multifit Bed Jack (5) Lift the head end of the bed. Fit the second Raiser (6) and lower the bed. Use the same method to fit a set of three Raisers.

**D**

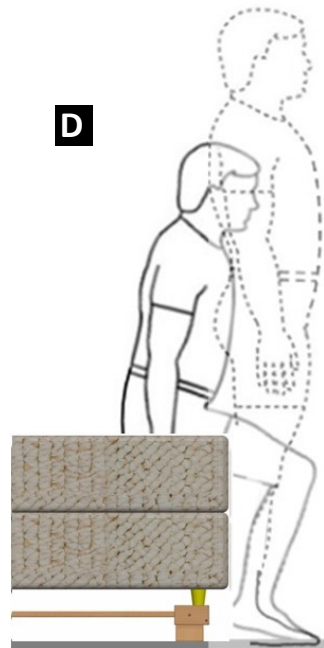

### Check:

Confirm the raised bed is safe and stable.  
Is getting in and out of the bed easier?  
Was the bed raised to the correct height?  
Does the user look confident?

**Maintenance** - check monthly the stability of the raised Bed. Repeat steps A,B,C,D if required.

**Cleaning** - with damp cloth and non-abrasive detergent, clean and dry thoroughly.

**Demonstrations** - see our stand at the next Show Your Ability expo.

**Parts replacement** - refer to [www.multifit.co.nz](http://www.multifit.co.nz) for a complete parts list.

**Our Mission** - is to provide you with the safest, most cost-effective raiser to meet your needs.



*Scan for more info*



Made with pride  
In New Zealand  
for over 25 years



**Multifit Hospital Supplies Ltd** © 2020  
Ph: 0800 102 803, E: [reception@multifit.co.nz](mailto:reception@multifit.co.nz)  
Website: [www.multifit.co.nz](http://www.multifit.co.nz) 230720