

Sofa Raiser MF 2/1

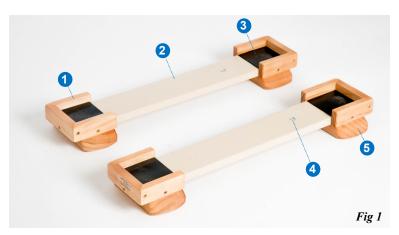

(1) Perimeter lip (2) Telescopic Section (3) 90x90 Rubber Pads (4) Bolt & Wing nut (5) Splayed feet

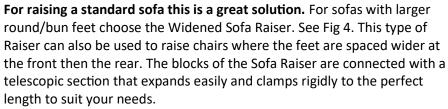

- Unique clamping action holds the Raiser tightly to a fixed size.
- **Sofa stays in place** 20mm perimeter lip holds the Raiser to the chair.
- Extra stability is provided with Splayed feet
- Safe Independently tested to ensure your safety.
- Easy to fit Telescopic adjustment ensures you get the perfect fit easily
- **Re Issuable** Robust construction enables the Raiser to be re issued.
- **Cost effective** Raisers sold 25 years ago are still in service today, making them very long lasting and cost-effective.

SPECIFICATIONS


Max user weight		380Kg (+ the furniture)		
lip around perimeter		20mm		
Pad size		90 x 90mm		
Sofa Raiser Set of 2, Also called Chair/Bed Troughs				
Height	Size	Supplier Code	Order Code	JDE#
150mm	624 mm to 900 mm	MF 2/1-150 / 2	K3-150/2	760739
100mm		MF 2/1-100 / 2	K3-100/2	760721
75mm		MF 2/1-75 / 2	K3-75/2	760755
50mm		MF 2/1-50 / 2	K3-50/2	760747

Optional: non slip rubber pads stop raiser slipping on polished timber floors **Safety First** - Raiser must match the chair type. See www.multifit.co.nz **Tested** - to ensure stability of the raised furniture. This safeguards the user.


Guarantee

- You will receive any List Item within five working days or we will treat you morning tea.
- Our equipment will fit or we will customise one for you that will.



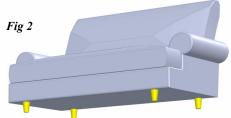

Raiser will fit this type of sofa

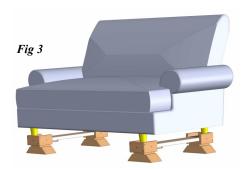

Raiser fitted to a sofa





Sofa with low





Made with pride In New Zealand for over 25 years



Multifit Hospital Supplies Ltd © 2020 Ph: 0800 102 803, E: reception@multifit.co.nz Website: www.multifit.co.nz 200720


Sofa Raiser MF 2/1

Fitting Instructions




The following method ensures the Raiser is fitted correctly, safely and with the minimum of effort. May require two persons to fit.




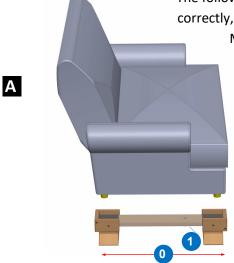

Adjust the Sofa Raiser in length (0) so the lip of the Raiser fits snugly over the feet of the sofa. Tighten the wing nut bolt (1)

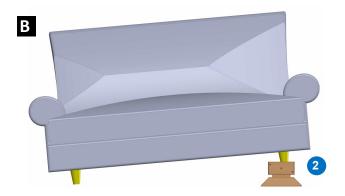


Fit one Raiser centrally under one end of the sofa (2)

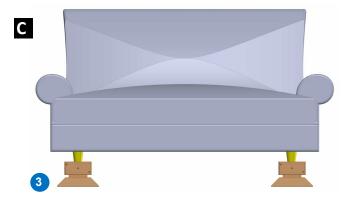

(3) At the other end, fit the second Sofa Raiser centrally under the feet of the sofa.

Checks:

Confirm the raised sofa is safe and stable. When getting out of the sofa your feet are flat on the floor and your bottom approximately level with the underside of your knees. Is sitting on and getting off the sofa easier? Is the comfort acceptable?

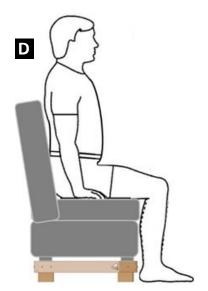

Maintenance - check monthly the stability of the raised sofa. Repeat steps A,B,C,D if required.

Cleaning - with damp cloth and non-abrasive detergent, clean and dry thoroughly.

Demonstrations - see our stand at the next Show Your Ability expo.

Parts replacement - refer to www.multifit.co.nz for a complete parts list.

Our Mission - is to provide you with the safest, most cost-effective raiser to meet your needs.



Scan for more info



