

Divan Sofa Raiser MF 2/5

Suits Sofas with no feet, or very low feet (Fig2)

- (1) Perimeter lip
- (2) 90x90 Rubber Pads
- (3) Cord
- (4) Cleat
- (5) Telescopic Section
- (6) Bolt & Wing nut
- (7) Splayed feet

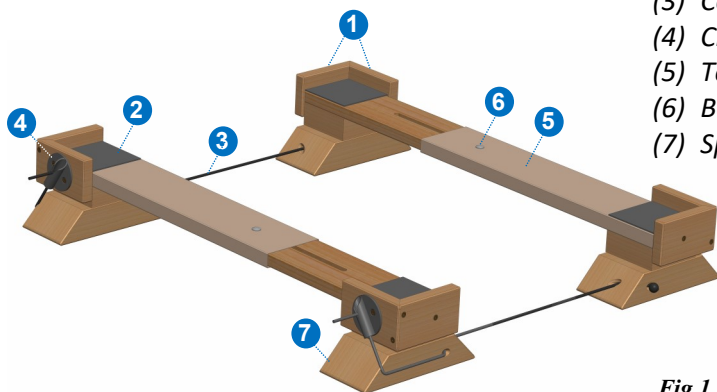


Fig 1



Pg1
MultiFit

Get on & off your
Sofa with less effort

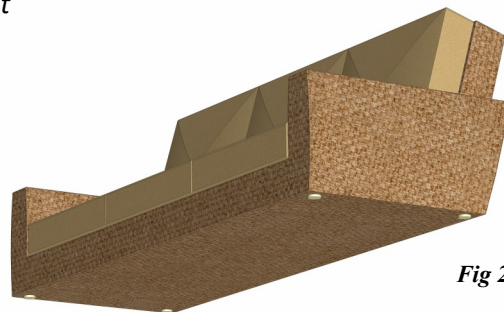


Fig 2

Divan Sofa Raisers have only two perimeter retaining lips (1).

This allows a Sofa with no feet, to sit down onto the Rubber Pad (2).
The Cord (3) spans between the Raisers and held in place by the Cleat (4).
The telescopic section (5) expands easily and clamps the Raiser rigidly to the perfect length to suit the width of your Sofa.

- **Unique clamping action** - holds the Raiser tightly to a fixed size.
- **Sofa stays in place** - 20mm perimeter lip holds the Raiser to the Sofa.
- **Extra stability** is provided with Splayed feet
- **Safe** - Independently tested to ensure your safety.
- **Easy to fit** - Telescopic adjustment ensures you get the perfect fit easily
- **Re Issuable** - Robust construction enables the Raiser to be re issued.
- **Cost effective** - Raisers sold 25 years ago are still in service today, making them very long lasting and cost-effective.

Raiser will fit this type of Sofa



Fig 3

MF 2/5 Raiser fitted to a Sofa

SPECIFICATIONS

Max user weight	380Kg (+ the furniture)
lip around perimeter	20mm
Pad size	90 x 90mm

Divan Sofa Raiser Set of 2. Also called Sofa or Chair Troughs

Height	Size	Supplier Code	Order Code	JDE#
150mm	624 mm to 900 mm	MF 2/5-150 / 2	CHRWYMF2562N	EK948
100mm		MF 2/5-100 / 2	CHRWYMF2542N	EK947
75mm		MF 2/5-75 / 2	CHRWYMF2532N	EK946
50mm		MF 2/5-50 / 2	CHRWYMF2522N	EK945

Optional: non slip rubber pads stop Raiser slipping on polished timber floors

Safety First - the Raiser must match the Sofa type. See www.multifit.co.nz

Tested - to ensure stability of the raised furniture. This safeguards the user.

Guarantee

- You will receive any List Item within five working days or we will treat you morning tea.
- Our equipment will fit or we will customise one for you that will.

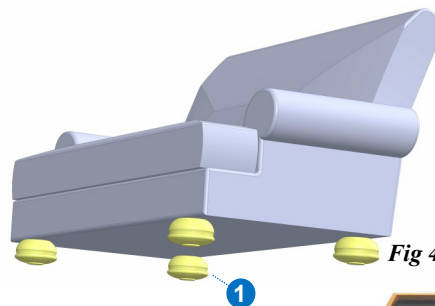


Fig 4

Larger Feet (1) ?

The Sofa Raiser Widened
MF 2/1W can be used.



Fig 5

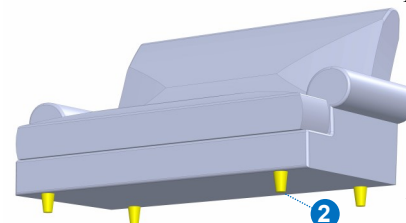


Fig 6

Normal feet (2) ?

The Sofa Raiser MF 2/1
can be used.



Fig 7



Made with pride
In New Zealand
for over 25 years



MultiFit Hospital Supplies Ltd © 2021
Ph: 0800 102 803, E: reception@multifit.co.nz
Website: www.multifit.co.nz 250421

Divan Sofa Raiser MF 2/5

Fitting Instructions




Pg2

MultiFit

Get on and off your Sofa with less effort.


The following method ensures the Raiser is fitted correctly, safely and with the minimum of effort. Take care when lifting the Sofa. May require two persons to fit.

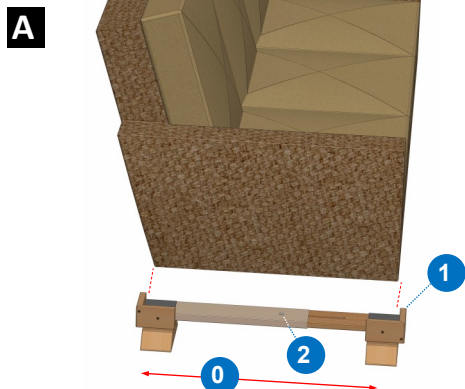

Adjust the Raiser in length (0) so the perimeter lip of the Raiser (1) fits snugly over the width of the Sofa. Tighten the wing nut bolt (2)




Fit a Raiser under one end of the sofa (3)

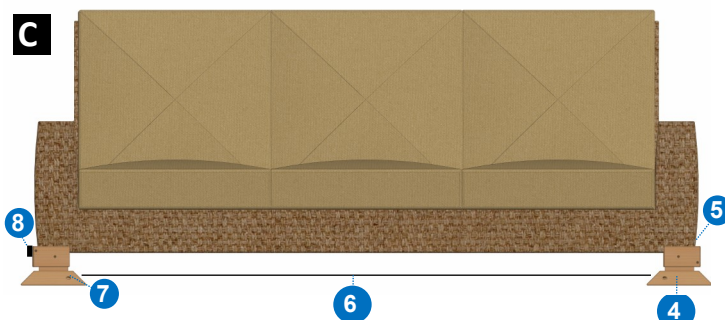

Fit the second Raiser (4). Make sure the perimeter lips of both Raisers are hard against the corners of the Sofa (5). Using the cord (6) Connect the two Raisers by threading the cord through the hole (7) in the foot of the Raiser. Tighten the cord and fix into the black plastic cleat (8)

Confirm the raised Sofa is safe and stable. When getting out of the Sofa your feet are flat on the floor and your bottom is approximately level with the underside of your knees. Is sitting on and getting off the Sofa easier? Is the comfort acceptable?

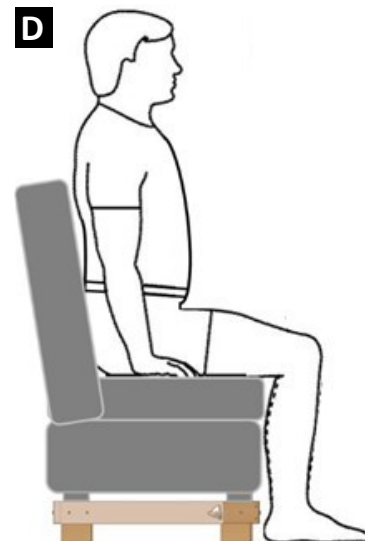

Checks: If abrupt or unforeseen movement occurs, such as the user falling onto the raised Sofa, then the Raisers will need to be checked to confirm they remain fitted as described above. The Raisers are only suitable for the situation described above.

All castor wheels must be supported by a caster cup.

Warning: potential trip hazard if any part of the Raiser protrudes past the perimeter of Sofa.

Maintenance - check monthly the stability of the raised Sofa. Repeat steps A,B,C,D if required.

Cleaning - with damp cloth and non-abrasive detergent, clean and dry thoroughly.

Demonstrations - see our stand at the next Show Your Ability expo.

Parts replacement - refer to www.multifit.co.nz for a complete parts list.

Our Mission - is to provide you with the safest, most cost-effective raiser to meet your needs.



Scan: more info



Made with pride
In New Zealand
for over 25 years



MultiFit Hospital Supplies Ltd © 2021
Ph: 0800 102 803, E: reception@multifit.co.nz
Website: www.multifit.co.nz 250421