

# Multifit LegLifter Mk3L

## Gentle bedside assistance for the elderly and disabled



NEW  
DESIGN FOR  
MULTI-LEVEL  
BEDS

**Increased independence for patients,  
easier management for health professionals**

The Multifit LegLifter is an appliance designed to provide independence for elderly and disabled people who are unable to lift their legs from a bedside sitting position onto the bed. Depending on the level of disability, the task of getting into bed safely can be performed with little or no assistance from another person.

### Easy to use

With simple push-button commands, the Multifit LegLifter gently carries the legs from the side of the bed to the prone position on the bed surface, then returns the leg-rest to its bedside position.

### Smooth, silent action

Precision-engineered componentry and robust construction ensures quiet performance and safe, gentle operation.



*Easy operation and safe, friendly ergonomics make the Multifit LegLifter an ideal bedside companion.*

### Adaptable for various domestic bed designs

The LegLifter Mk3L is designed for installation to typical domestic innerspring, slat or frame beds. It is adjustable to suit preferred seating heights from 300 mm to 700 mm. Adaptations to fit profiling beds can be supplied on request.

### Installs in minutes

Installation for domestic beds and many profiling beds takes from five to ten minutes. Allen-key tools are supplied for assembly.

### Compact, portable and slim

Compact dimensions and 9 kg weight make LegLifter Mk3L easy to carry and transport, even in a small vehicle. The slim profile of the installed unit fits close beside the bed when not in operation.

### High performance and durability

Multifit LegLifter Mk3L has been durability tested successfully to 10,000 cycles.

### Low maintenance, easy care

Innovative design and sturdy engineering offer easy cleaning and low maintenance.

### Mk3L features include:

- New design for low-level bed frames
- Access under bed for bed-table trolley
- Options for standard or wide leg-rests to suit specific patient needs
- Synchronised lift action to suit needs of patients with diminished balancing ability
- Handset light to show when the leg-rest is lowered safely to the bedside
- Refined ergonomics, slimmer profile
- Optional battery backup



*Single-leg design provides space for bed-table trolley access and standing space for bedside tasks.*

# Multifit LegLifter Mk3L

## Specifying and ordering the Multifit LegLifter for individual requirements

The unique Multifit LegLifter design lends itself to many types of bed frame styles, both domestic and profiling, to suit access from either side of the bed.



Factory fitted for left side of bed



Factory fitted for right side of bed

The Multifit LegLifter utilises a slim, strong stabiliser which fits under the mattress to ensure sturdy installation and smooth operation. It is equipped for fitting to innerspring, frame, slat and box bed-bases. Some profiling bed frames may require custom-designed fittings to suit the type and width of bed frame. When making enquiries or ordering, specify:

- Domestic frame type and width, **or**
- Profiling bed brand and model
- Access preference from left or right side of bed, viewed from the typical prone position in the bed:

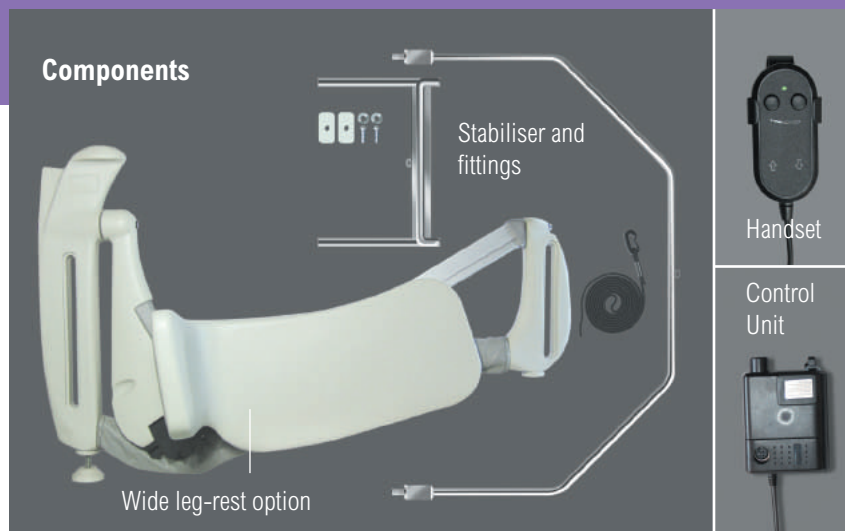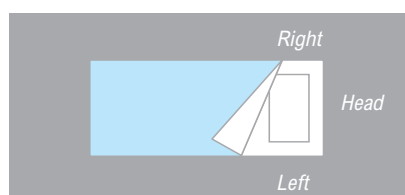

LegLifter unit dimensions	Width 1050 mm	Height 660 mm
Height adjustment range	300 mm to 700 mm	
Total weight including packaging for transit	16 kg	
Weight of LegLifter unit	9 kg	
Installation time for standard domestic bed	From 5 to 10 minutes	
Maximum leg-load weight	Tested to 67 kg: legs must fit within leg-rest	
Distance from bedframe to platform of folded leg-rest	60 mm	

- An optional wider, interchangeable leg-rest is available to suit patients with swollen or heavy legs. Specify standard or wide leg-rest when ordering:




### Safety features

- Low 28 V power to the LegLifter
- Automatic sensor turns motor off in the event of obstruction behind the footrest
- Telescopic support prevents leg-rest lowering under power against obstruction
- Rounded edges prevent injury
- Handset light turns off when leg-rest is fully and safely lowered beside the bed.

### Distributed by:

### See the LegLifter Mk3L in action!

To view a brief video and to download more information, visit [www.multifit.co.nz](http://www.multifit.co.nz)

Designed and manufactured by:   
 Multifit Hospital Supplies Ltd  
 PO Box 748 Whakatane 3158, New Zealand