

Leg Lifter Transfer rail installation

instructions.

20.5.13

The rail will fit both the MK3 and the Mk3L leg lifters.

1/. The transfer rail is configure left or right hand side of the bed. So ensure you have ordered the rail to match the leg lifter configuration you have.

2/. Fit the stability bar between the mattress and the base of your bed and remove the fixing bolts and washers

3/. Sit the leg Lifter at the bedside.

4/. On the Transfer rail—Remove the fixing bolts and washers.



5/. Fit the Transfer rail up to the handle end slot in the leg lifter.

6/. Fix the bolts through the slot and into the transfer rail and tighten with the square washers horizontally positioned.



7/. Fit the leg lifter against the bed. Fit and tighten the bolts and washers into the stability bar so that the end of the leg lifter 25mm below the top surface of the mattress.

8/. Adjust the adjustable foot of the transfer rail so the foot is on the floor and the rail supports the leg lifter in a vertical position.

